

Universal Surround Toilet Frame


SETUP INSTRUCTIONS

Congratulations on the purchase of your Q-assist Universal Surround Toilet Frame.

Included in the box


Please ensure that all parts are present before attempting to setup the device.

A	2 x Frame Sides
B	2 x Connecting Tubes
C	4 x C-Clips

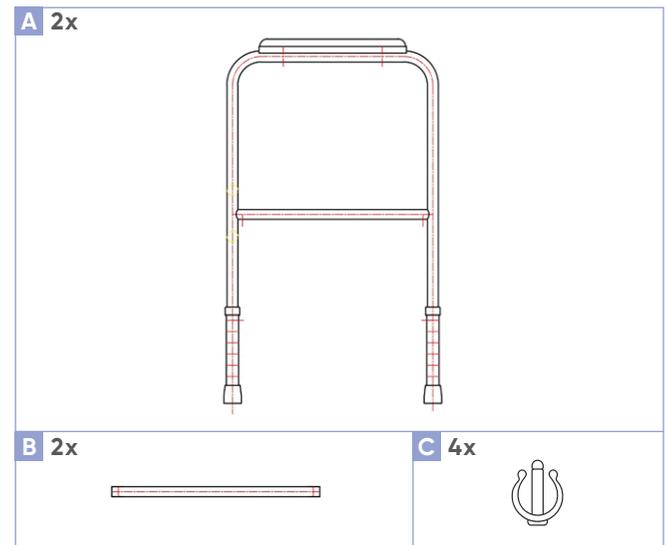


Step 1:

Connect the first side of the frame together with the connecting tubes by inserting the tube into the frame, line up the holes and insert the C-Clip to secure together. The product should initially be set up at the widest setting.

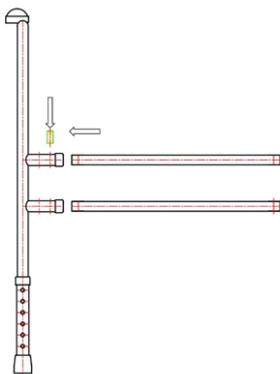

Step 2:

Repeat step 1 to finish set up, ensuring that the frame is sturdy and secure once all connected together.

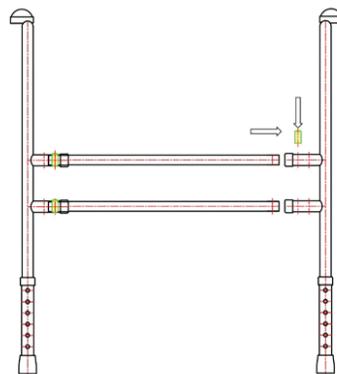

Step 3:

Remove C-Clips to adjust the frame to fit your toilet. Height can be adjusted by depressing the pin and extending or shortening legs as required. Ensure all C-Clips are securely in place after all adjustments.

