

# 360mm Compact Step Ramp

## GUIDE FOR INSTALLATION AND SAFE USE

### What is a Step Ramp?

A step ramp is a valuable tool for improving accessibility and mobility, particularly for overcoming low to medium-height steps. By providing a smooth and safe transition over steps or thresholds, step ramps help individuals with mobility challenges navigate their environment more easily, whether at home, in public spaces, or while traveling.

All instructions contained within this document should be used alongside any information provided by your occupational therapist or medical professional. This product should only ever be used as is described in this document.

### Installation guide

- Place ramp in the desired location.
- Locate holes on top and base, using a marker, mark the holes.
- Remove ramp and predrill all holes.
- Place ramp back into place and fix into place using the provided screws.

### Safe use

#### For manual wheelchair users


- **Ascend or Descend Carefully:** If going up, push the wheelchair slowly up the ramp. If descending, go down slowly, either rolling down under control or, if necessary, having an assistant assist in lowering the chair.
- **Safety:** Always use the ramp slowly to maintain control, especially if the surface is wet or slippery. Consider having an assistant help ascend and descend the ramp.
- **Guide the Wheels:** Make sure the wheelchair wheels are centered on the ramp while using it.

#### For powered wheelchair users

- **Activate the Motor:** If using a powered wheelchair, engage the motor to move the wheelchair up or down the ramp.
- **Steady Pace:** Ensure the speed is controlled and steady, especially when descending, to avoid losing control.
- **Assistant Help:** Sometimes it's helpful to have an assistant nearby, especially when using the ramp for steeper inclines.

### Contents of box

<b>A</b>	1 x 360mm Compact Step Ramp
<b>B</b>	4 x Fixing Screws

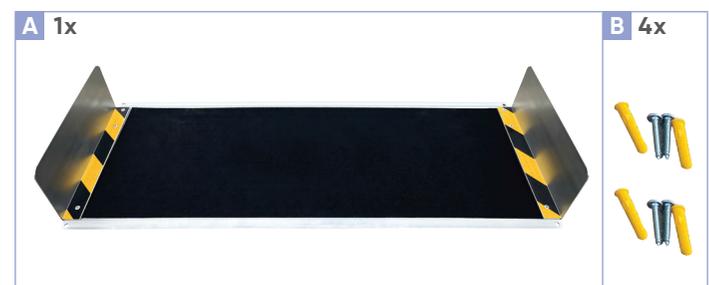

### Max set up height and loading heights - Maximum load 360kg

Ramp Length	Minimum Loaded Height	Maximum Loaded Height
360mm	30mm	75mm
AS 1428.1 Gradient 1 in 10	36mm	36mm

### Safety & maintenance

- The folding ramp should be regularly checked to make sure that the product is operating nominally and free from damage.
- This ramp does not have any serviceable parts. The entire ramp needs to be replaced.

### Ramp safety

- **Never load more than 360kg on the Step Ramp.**
- **Never exceed the Maximum height limits and loading ratios.**
- **Never attempt to move the product while under load.**
- **Always ensure the top of the ramp is securely placed prior use.**
- **Always thoroughly check the product for any signs of damage prior to use.**
- **Never use the product if there are any signs of damage.**
- **When using a mobility device always follow the devices manufacturers recommended instructions for use with ramps.**
- **Never use this product in anyway other way other than the intended use disclosed in this document.**