# evekare

## INSTALLATION GUIDE ABS GRAB RAIL

#### IMPORTANT

- For cavity walls, the grab rail must be fixed to the internal wall stud or a solid framing structure. This may require the grab rail to be installed on an angle to match the gap between the studs.
- The included fixings are for use with solid walls only. It is the installer's responsibility to ensure that the correct fixings are used to suit the wall type.

#### INSTALLATION

- 1. Unclip the bracket covers to expose the bracket holes.
- 2. Place the grab rail in the desired position and mark the holes for drilling.
- 3. Using an appropriate drill bit (i.e. masonry bit for tiles), drill holes to suit the wall plugs.
- 4. Insert wall plugs.
- 5. Line up the grab rail with the drilled holes and install the screws.
- 6. Push the bracket covers onto the brackets until they clip into place.













### GLOW-IN-THE-DARK MODELS

To achieve sufficient luminance, the grab rail must be exposed to ultra violet light, such as the sun or a fluorescent light source.

#### CARE AND MAINTENANCE

- · Clean with a soft cloth and warm soapy water.
- · Do not use harsh detergents, corrosive chemicals, or abrasive cleaners.