

1525mm Telescopic Wheelchair Ramp

GUIDE FOR INSTALLATION AND SAFE USE

What is a Telescopic Wheelchair Ramp?

A Telescopic Wheelchair Ramp is a portable, lightweight ramp designed to help people in wheelchairs or mobility devices safely navigate steps, curbs, or other raised surfaces. These ramps fold in half for easy storage and transport, making them especially convenient for individuals who need a ramp only occasionally or on the go.

All instructions contained within this document should be used alongside any information provided by your occupational therapist or medical professional. This product should only ever be used as is described in this document.

Safe use

1. Prepare the area:

- **Clear the area:** Make sure the area where you'll be using the ramp is free of obstacles, such as debris, furniture, or uneven surfaces.
- **Check the surface:** Ensure the ground or surface the ramp is going to rest on is stable and level. This is especially important for the bottom of the ramp to prevent it from slipping.

2. Extend the ramp:

- **Extend carefully:** Extend both telescopic ramps by depressing brass pins. Make sure both sections are fully extended, and brass pins are securely locked in place.
- **Position the ramp:** Place the telescopic ramps at the edge of the step, threshold, or elevated surface.
- **Check for stability:** Ensure the ramp is properly locked into place so that it doesn't collapse while in use.

3. Position the ramp:

- **Top edge:** The top of the ramp should be placed securely against the raised surface (e.g., a step or vehicle). If the ramp is meant to be used for loading a wheelchair into a car, make sure the ramp is flush with the vehicle's door or lift.
- **Bottom edge:** The bottom of the ramp should rest on a flat, stable surface. If it's going into a vehicle or another raised surface, make sure it's properly supported.
- **Width:** Ensure that the width is correct for the mobility device being used.

4. Using the ramp for manual wheelchair users:

- **Ascend or descend carefully:** If going up, push the wheelchair slowly up the ramp. If descending, go down slowly, either rolling down under control or, if necessary, having an assistant assist in lowering the chair.
- **Safety:** Always use the ramp slowly to maintain control, especially if the surface is wet or slippery. Consider having an assistant help ascend and descend the ramp.
- **Guide the wheels:** Make sure the wheelchair wheels are centred on the ramp while using it.



5. After Use:

- **Fold the ramp back:** When you're done, carefully fold the ramp back into its compact form.
- **Store safely:** Store the ramp in a safe location to prevent tripping hazards and to protect it from damage.

Max set up height and loading heights - Maximum load 300kg

Ramp Length	Max Height - 1 in 6 Gradient	Max Loaded Height
1525mm	254mm	254mm

Safety & maintenance

- The folding ramp should be regularly checked to make sure that the product is operating nominally and free from damage.
- This ramp does not have any serviceable parts. The entire ramp needs to be replaced.

Ramp safety

- **Never load more than 300kg on the telescopic ramp.**
- **Never exceed the Maximum height limits and loading ratios.**
- **Never attempt to move the product while under load.**
- **Always ensure the top of the ramp is securely placed prior use.**
- **Always ensure that brass pins are locked into place.**
- **Always thoroughly check the product for any signs of damage prior to use.**
- **Never use the product if there are any signs of damage.**
- **When using a mobility device always follow the devices manufacturers recommended instructions for use with ramps.**
- **Never use this product in anyway other way other than the intended use disclosed in this document.**