

Shower Chair

SETUP INSTRUCTIONS


Congratulations on the purchase of your Q-assist Shower Chair.

Type B product for domestic use only.

Included in the box


Please ensure that all parts are present before attempting to setup product.

A	1 x Seat
B	1 x Frame
C	4 x Chair Legs
D	4 x Stainless Steel Screw
E	8 x Nylon Washers

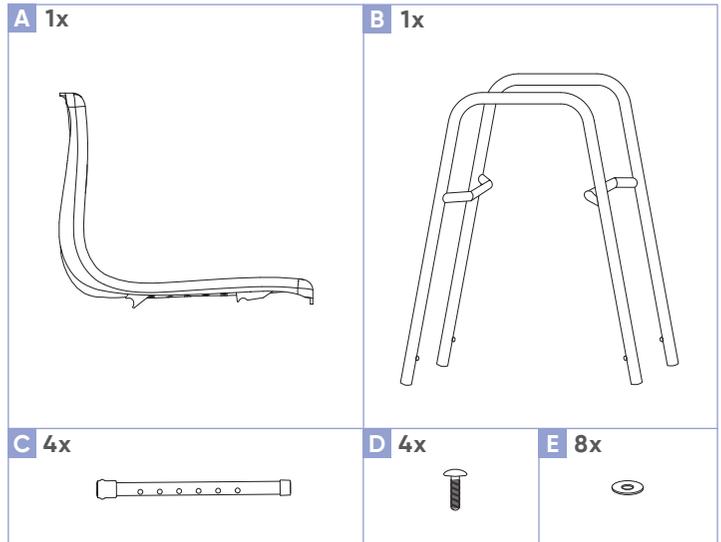

Step 1:

Fit the seat to the frame using the four stainless screws, each should be fitted through two nylon washers and inserted into the seat up through the frame. Do not over-tighten, hand tight is sufficient.

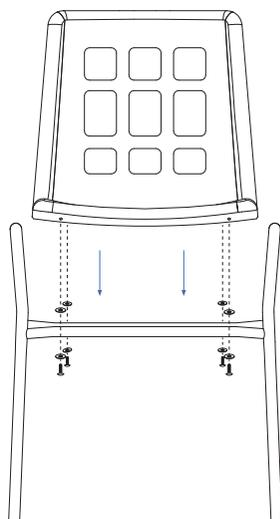

Step 2:

Fit an extension leg onto each frame leg, ensure brass buttons fully lock into place.

