

Portable Stand Assist Aid


SETUP INSTRUCTIONS

Congratulations on the purchase of your Q-assist Portable Stand Assist Aid.

Included in the box


Please ensure that all parts are present before attempting to setup product.

A	1 x Base
B	2 x Leg Frames
C	1 x Arm Rest
D	1 x Parallel Tube
E	1 x Storage Pouch
F	6 x Screws

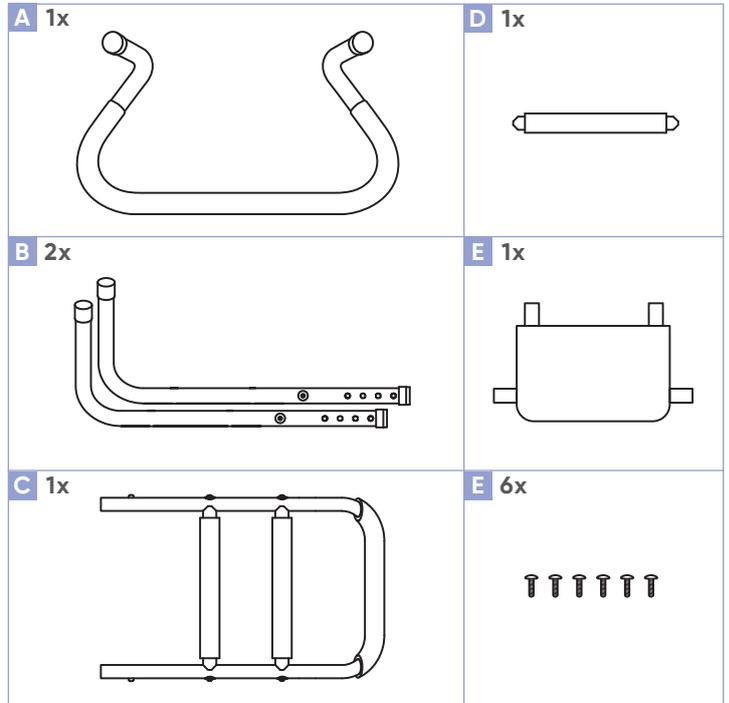


Step 1:

Attach the leg frames and the base together by fastening the screws into the corresponding holes.

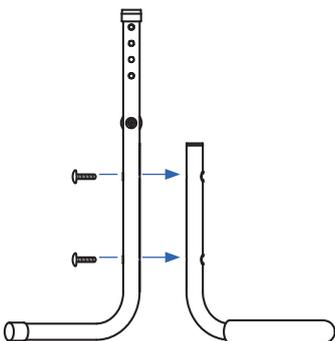

Step 2:

Attach the parallel tube to the top of the attached base and leg frames using 2 x screws.

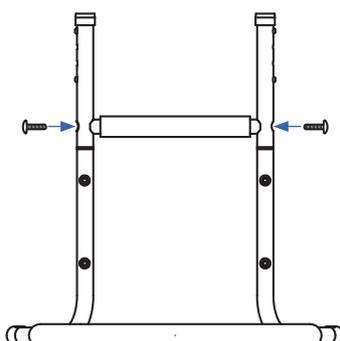

Step 3:

Insert the armrest into the base of the portable stand. Make sure that the brass pin locks into place.

