



Meets Australian Standard AS 3856

Q-Assist 900mm Portable Folding Wheelchair Ramp with Carry Bag

QEP Item NO: QA09

Bunnings Item NO: 0648143

The Q-Assist 900mm Portable Folding Wheelchair Ramp makes raised surfaces accessible for individuals using mobility aids including wheelchairs, scooters and walkers.


Designed to be taken on the go, this innovative ramp features a single-fold design with a carry handle and bag for easy transport and storage.

The portability of this ramp makes it a convenient option for individuals who need a ramp only occasionally or when they are on the go. The Q-Assist 900mm Portable

Folding Wheelchair Ramp can be used for a range of raised surfaces including steps, thresholds, curbs and even into vehicles.

Made from lightweight, durable aluminium for easier manoeuvrability the Q-Assist 900mm Portable Folding Wheelchair Ramp is for both internal and external use.

For user comfort and safety, the ramp features 75mm high kerb rails and a non-slip surface. To clearly identify the usable ramp area, it also features high contrast tape along both sides of the ramp.

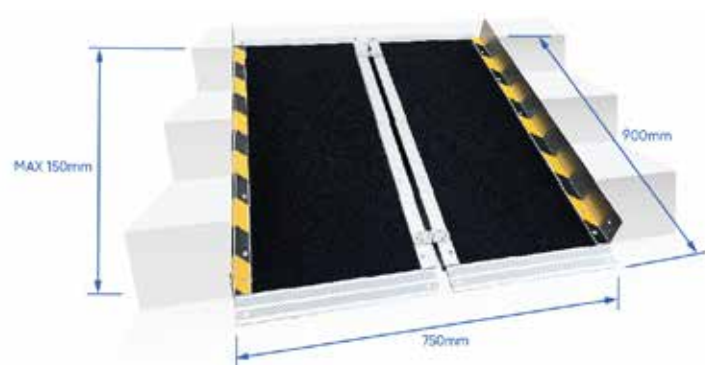

The Q-Assist 900mm Portable Folding Wheelchair Ramp features a weight rating of 400kg and is 750mm wide to accommodate both manual and powered wheelchairs. The ramp meets Australian Standard AS 3856 and features a max loaded height of 150mm and a 1 in 6 gradient max height of 150mm. For more information on set up and safe use please refer to the Guide for Installation and Safe Use document.

This ramp is 900mm long, for a longer portable ramp see the Q-Assist 1800mm Portable Folding Wheelchair Bag with Carry Bag.

- An essential assistive device for individuals with disabilities, the elderly and those who are recovering from surgery as it enables safe access to raised areas for individuals using mobility aids
- Suitable for both manual and motorised wheelchairs, scooters and walkers
- Use on curbs, raised thresholds, steps and for vehicle access
- Single-fold design with carry handle and bag for easy transport and storage
- Underside rubber pads assist with secure attachment and minimise noise
- High contrast tape assists with clearly identifying the usable ramp area
- 75mm kerb rails for added security while using the ramp
- Perfect for internal and external use
- Nonslip surface
- Made from lightweight, durable aluminium

900mm Portable Folding Wheelchair Ramp

GUIDE FOR INSTALLATION AND SAFE USE

What is a Folding Wheelchair Ramp?

A Folding Wheelchair Ramp is a portable, lightweight ramp designed to help people in wheelchairs or mobility devices safely navigate steps, curbs, or other raised surfaces. These ramps fold in half for easy storage and transport, making them especially convenient for individuals who need a ramp only occasionally or on the go.

All instructions contained within this document should be used alongside any information provided by your occupational therapist or medical professional. This product should only ever be used as is described in this document.

Safe use

1. Prepare the area:

- **Clear the area:** Make sure the area where you'll be using the ramp is free of obstacles, such as debris, furniture, or uneven surfaces.
- **Check the surface:** Ensure the ground or surface the ramp is going to rest on is stable and level. This is especially important for the bottom of the ramp to prevent it from slipping.

2. Unfold the ramp:

- **Position the ramp:** Place the folded ramp at the edge of the step, threshold, or elevated surface. Ensure the ramp is aligned so that the sloped side will face the ground.
- **Unfold carefully:** Carefully open the ramp along its hinges. Make sure both sections are fully extended and secure.
- **Check for stability:** Ensure the ramp is properly secured into place so that it doesn't collapse while in use.

3. Position the ramp:

- **Top edge:** The top of the ramp should be placed securely against the raised surface (e.g., a step or vehicle). If the ramp is being used for loading a wheelchair into a car, make sure the ramp is flush with the vehicle's door or lift.
- **Bottom edge:** The bottom of the ramp should rest on a flat, stable surface. If it's going into a vehicle or another raised surface, make sure it's properly supported.


4. Using the ramp:

For manual wheelchair users

- **Ascend or descend carefully:** If going up, push the wheelchair slowly up the ramp. If descending, go down slowly, either rolling down under control or, if necessary, having an assistant assist in lowering the chair.
- **Safety:** Always use the ramp slowly to maintain control, especially if the surface is wet or slippery. Consider having an assistant help ascend and descend the ramp.
- **Guide the wheels:** Make sure the wheelchair wheels are centred on the ramp while using it.

For powered wheelchair users

- **Activate the motor:** If using a powered wheelchair, engage the motor to move the wheelchair up or down the ramp.

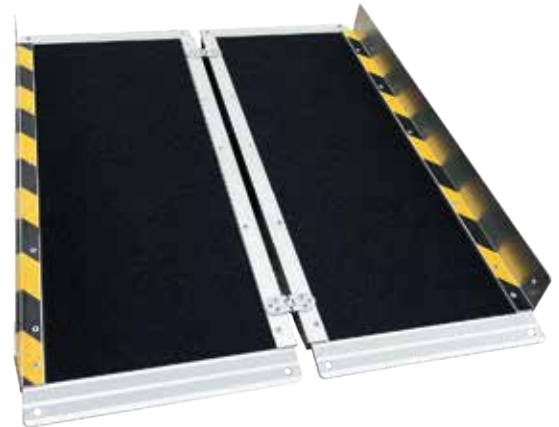

- **Steady Pace:** Ensure the speed is controlled and steady, especially when descending, to avoid losing control.
- **Assistant Help:** Sometimes it's helpful to have an assistant nearby, especially when using the ramp for steeper inclines.

5. After use:

- **Fold the ramp back:** When you're done, carefully fold the ramp back into its compact form.
- **Store safely:** Store the ramp in a safe location to prevent tripping hazards and to protect it from damage.

Max set up height and loading heights - Maximum load 400kg

Ramp Length	Max height - 1 in 6 Gradient	Max Loaded Height
900mm	150mm	150mm

Safety & maintenance

- The folding ramp should be regularly checked to make sure that the product is operating nominally and free from damage.
- This ramp does not have any serviceable parts. the entire ramp needs to be replaced.

Ramp safety

- **Never load more than 400kg on the folding ramp.**
- **Never exceed the Maximum height limits and loading ratios.**
- **Never attempt to move the product while under load.**
- **Always ensure the top of the ramp is securely placed prior use.**
- **Always thoroughly check the product for any signs of damage prior to use.**
- **Never use the product if there are any signs of damage.**
- **When using a mobility device always follow the devices manufacturers recommended instructions for use with ramps.**
- **Beware of entrapment when folding the ramp.**
- **Never use this product in anyway other way other than the intended use disclosed in this document.**