

INSTALLATION GUIDE FOLDING SHOWER CHAIR

IMPORTANT

Periodically check the fixings to ensure that they are secure and re-tighten if necessary. Do not use if any fixings are loose. Always check to ensure that the product is stable before use.


PRE-ASSEMBLY INSPECTION

Ensure all parts are present and thoroughly inspect for damage or defects. Damaged or defective parts may compromise structural integrity. If any damage or defects are found, do not continue with installation and return to the place of purchase.


PARTS



Seat Assembly x 1

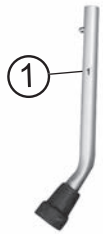


Short Legs x 2

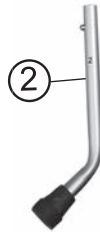


Long Legs x 2


INSTALLATION

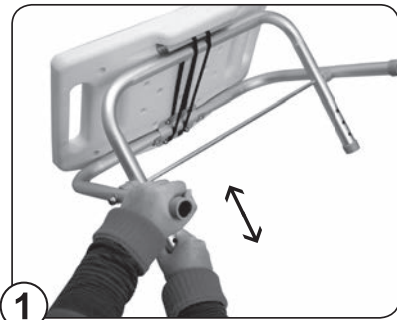


1 Unfold the seat assembly.

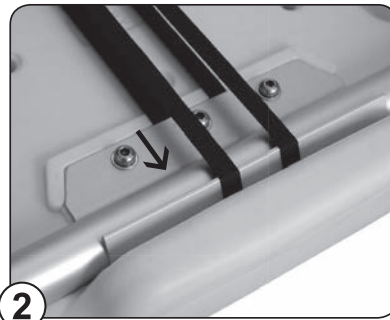


2 Push the tube into the seat bracket until it clicks into place.

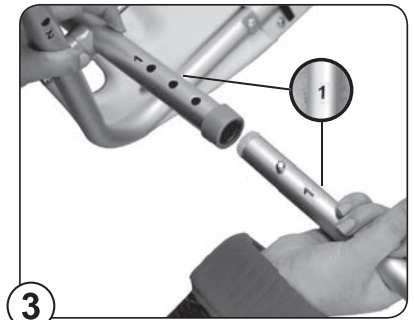


3 Insert the short legs into the frame. Match the leg numbers with the frame numbers.

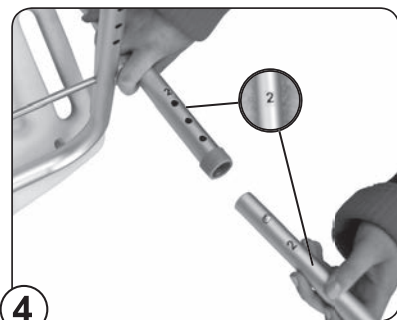

4 Insert the long legs into the frame. Match the leg numbers with the frame numbers.

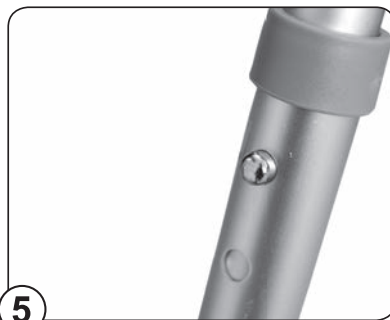

5 Ensure the pins are fully engaged and all legs are configured at the same height.

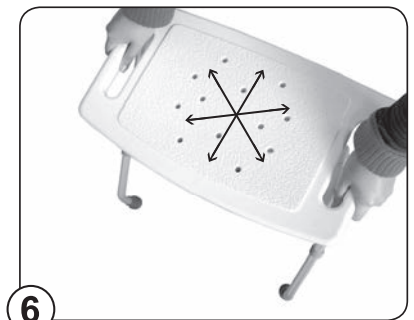

6 Check to ensure that the seat is stable before use.

CARE AND MAINTENANCE

- Clean with a soft cloth and warm soapy water.
- Do not use harsh detergents, corrosive chemicals, or abrasive cleaners.