

Portable Stand Assist Aid

GUIDE FOR INSTALLATION AND SAFE USE

What is a Portable Stand Assist Aid?

The Q-assist Portable Stand Assist Aid is used to help patients stand from sitting positions or transfer down to sitting from standing. It is a versatile device that can be used in a variety of settings throughout the home. It has a safe load of 136kg.

Installation prior to first use

All instructions contained within this document should be used alongside any information provided by your occupational therapist. This product should only ever be used for its intended purpose as described within this document.

Safe use

- 1. Prepare the Portable Stand Assist Aid:**
 - Adjust the Portable Stand Assist Aid to the most comfortable height for use. Height should be adjusted so that it is comfortable to use in both standing and sitting positions.
 - The area around the Portable Stand Assist Aid should be free from obstacles and obstructions.
 - Your occupational therapist will provide specific recommendations tailored to your circumstances for use of this product.
- 2. Safe Transfers:**
 - Make sure that the Portable Stand Assist Aid is positioned at an appropriate distance to ensure a safe and comfortable transfer.
 - The feet of the portable stand assist should be level, never attempt a transfer on an uneven surface.
 - Test the stability of the Portable Stand Assist Aid prior transfer.
 - Your occupational therapist will provide specific recommendations tailored to your circumstances for use of this product.
- 3. Weight Capacity:**
 - Q-assist Portable Stand Assist Aid has a safe working load of 136kg, never exceed this limit doing so risks product failure and injury.



Cleaning & maintenance

- Ensure Portable Stand Assist Aid is regularly cleaned with a mild disinfectant
- Fittings should be regularly checked to ensure that they do not become loose after normal operation.
- **Never** use if the Portable Stand Assist Aid sustains or appears to be damaged in any way, the product may require replacing.

Product safety

- Your occupational therapist will provide specific recommendations tailored to your circumstances for use of this product. Instructions contained within this document are designed to be used in conjunction with occupational therapist recommendations.
- Ensure that the surface around intended transfer is even, never attempt a transfer on an uneven surface.
- Always test stability prior to use.
- Never to be used by more than one person at a time.
- **Never load more than 136kg on the product.**
- **Never use the product if there are any signs of damage.**
- **Never use this product in anyway other way other than the intended use disclosed in this document.**