

Bath Board


SETUP INSTRUCTIONS

Congratulations on the purchase of your Q-assist Bath Board with swivel seat.

Included in the box


Please ensure that all parts are present before attempting to setup product.

A	1 x Bath Board
B	1 x Swivel Seat
C	1 x Transfer Handle
D	3 x Plastic Bolt (Circular)
E	4 x Plastic Bolt (Rectangular)
F	4 x Bath Board Feet
G	4 x Plastic Washer
H	1 x Metal Phillips Head Screw
I	1 x Metal Washer

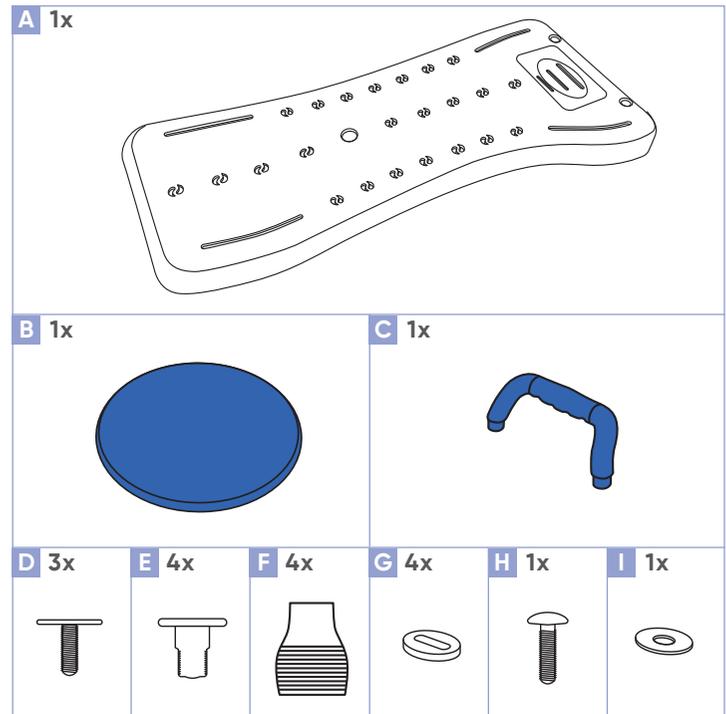


Step 1: Handle setup

Insert the handle into the two holes at the end of the Bath Board. The handle can be installed facing inward or outward according to the comfort of the user. Insert screws into each side of the handle and tighten until firmly secured.

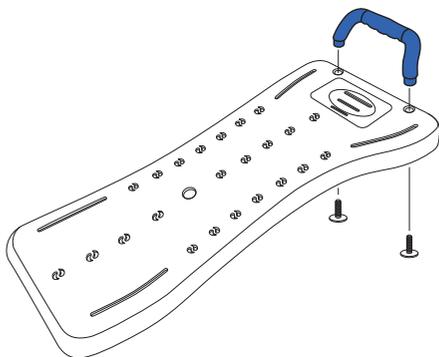

Step 2: Install adjustable feet

Using the rectangular plastic bolts and plastic washers, secure the four feet to the underside of the Bath Board. The bolt goes through the surface of the Bath Board, through a washer on the underside and then screws into the feet. Tighten until firmly secured.

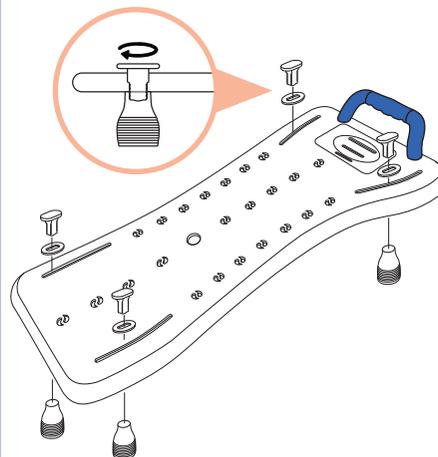

Step 3: Attach swivel seat

Place the blue swivel disc into the hole in the middle of the Bath Board. Whilst holding in place, use the circular bolt to secure in place from the underside. Tighten until secured in place but can still rotate. Do not over tighten as the disc will not swivel.

