



Heeve 1000mm Solid Rubber Ramp

VARIANTS

Size:

Heeve 1000mm Solid Rubber Ramp - 10mm x 40mm

Heeve 1000mm Solid Rubber Ramp - 15mm x 60mm

Heeve 1000mm Solid Rubber Ramp - 20mm x 80mm

Heeve 1000mm Solid Rubber Ramp - 30mm x 100mm

MAXIMUM WEIGHT CAPACITY: 1000kg

INSTALLATION INFORMATION:

1. Preparation:

Identify the threshold area where the ramp will be placed. Ensure that the surface is clean and free of debris to prevent slippage.

Measure the height of the threshold. The Heeve ramp is available in sizes ranging from 10mm to 30mm. Select the appropriate ramp height for your specific threshold.

2. Placement of the Ramp:

Place the ramp directly against the threshold or step. Ensure that the front lip of the ramp sits flush with the edge of the threshold for a smooth transition.

3. Check for Fit:

Ensure that the ramp is stable and flush with the ground. The weight of the solid rubber ramp should keep it in place without the need for adhesives or screws.

Modify the ramp, if necessary. Though the ramp is designed to fit most thresholds, it can be trimmed or adjusted slightly to better fit your specific area. Use a heavy-duty utility knife for minor alterations.

4. Test the Ramp:

Test the ramp by slowly rolling a wheelchair over it to ensure that the transition is smooth and that the ramp does not shift.

Check the traction by walking or rolling over the ramp with dry and wet conditions to confirm that the moulded drain channels are functioning properly.

SAFETY INFORMATION:

1. Weight Capacity: Do not exceed the 1000kg weight limit.
2. Stability Check: Ensure the ramp is placed on a flat, stable surface free of debris.
3. Wet Conditions: Exercise caution on wet surfaces to prevent slips.
4. Speed and Turns: Approach the ramp slowly and avoid sharp turns.
5. Regular Inspection: Check for wear and tear, replacing if damaged.
6. Proper Surfaces: Use only on appropriate thresholds, doorways, or small steps.
7. Modifications: Avoid major alterations that affect safety.
8. Indoor/Outdoor Use: Regularly check the ramp for dirt or debris outdoors.
9. Handling: Use proper lifting techniques due to the ramp's weight.
10. Clear Area: Keep the area around the ramp free of obstacles.
11. Supervision: Supervise first-time users to ensure safe navigation.

MAINTENANCE:

Easy steps to maintain your rubber kerb ramp:

1. Clean regularly using water and mild detergent to remove dirt and debris.
2. Ensure stability by checking the ramp's position periodically.
3. The ramp is weather-resistant, but for extended durability, it's advisable to inspect the ramp regularly for any signs of wear, particularly if it is used in outdoor environments, and replace it if necessary.

WARRANTY INFORMATION:

Heeve products come with a one-year warranty covering manufacturer defects, excluding damage from improper use, assembly, storage, or normal wear and tear.