

Solid Aluminium Threshold Ramp

GUIDE FOR INSTALLATION AND SAFE USE

What is a Threshold Ramp?

A Threshold Ramp is a type of mobility ramp specifically designed to bridge small height differences, such as those found at doorways, thresholds, or small steps. It improves accessibility for individuals using wheelchairs, scooters, walkers, or other mobility aids.

All instructions contained within this document should be used alongside any information provided by your occupational therapist or medical professional. This product should only ever be used as is described in this document.

Installation guide

- Place ramp in the desired location inside door leaf.
- Locate pre-drilled holes, using a marker, mark the holes.
- Remove ramp and pre-drill all holes.
- Place ramp back into place and fix into place using the provided screws.

For manual wheelchair users

- **Ascend or Descend Carefully:** If going up, push the wheelchair slowly up the ramp. If descending, go down slowly, either rolling down under control or, if necessary, having an assistant assist in lowering the chair.
- **Safety:** Always use the ramp slowly to maintain control, especially if the surface is wet or slippery. Consider having an assistant help ascend and descend the ramp.
- **Guide the Wheels:** Make sure the wheelchair wheels are centered on the ramp while using it.

For powered wheelchairs:

- **Activate the Motor:** If using a powered wheelchair, engage the motor to move the wheelchair up or down the ramp.
- **Steady Pace:** Ensure the speed is controlled and steady, especially when descending, to avoid losing control.
- **Assistant Help:** Sometimes it's helpful to have an assistant nearby, especially when using the ramp for steeper inclines.

Contents of box

A	1 x 280mm Threshold Ramp
B	2 x Fixing Screws

A 1x		B 4x	
-------------	------------------------------------------------------------------------------------	-------------	-------------------------------------------------------------------------------------

Max set up height and loading heights - Maximum load 360kg

Ramp Length	Minimum Loaded Height	Maximum Loaded Height
280mm	10mm	35mm
AS 1428.1 Gradient 1 in 8	10mm	35mm

Ramp safety

- **Never load more than 360kg on the Threshold Ramp.**
- **Never exceed the Maximum height limits and loading ratios.**
- **Never use the product if it has not been secured as described in this document.**
- **Always thoroughly check the product for any signs of damage prior to use.**
- **Never use the product if there are any signs of damage.**
- **When using a mobility device always follow the devices manufacturers recommended instructions for use with ramps.**
- **Never use this product in anyway other way other than the intended use disclosed in this document.**