

Bathroom Swivel Turnplate

GUIDE FOR INSTALLATION AND SAFE USE

What is a Bathroom Swivel Turnplate?

The Q-Assist Bathroom Swivel Turnplate assists with the transfer of seated patients with limited lower limb mobility who cannot move their feet during assisted transfers. Safe working load of 150kg.

Installation prior to first use

All instructions contained within this document should be used alongside any information provided by your occupational therapist. This product should only ever be used

NOTE: This product requires a carer to assist patient with use it should never be used without assistance.

Safe use

1. Preparation:

- Before using the Bathroom Swivel Turnplate, ensure the bathroom floor is dry to prevent slipping.
- Place turn plate on the ground so that it sits flat and it is in the idea location for the transfer.

2. Using Bathroom Swivel Turnplate:

- Slowly approach the Bathroom Swivel Turnplate, using any of your normal aids.
- With the assistance of your care giver place your feet evenly in the centre of the Bathroom Swivel Turnplate.
- With the assistance of your care giver slowly raise to your feet,
- Allow your care giver to turn you on the Bathroom Swivel Turnplate, do not attempt this action on your own.
- Once you have been repositioned, reverse the above process to slowly get off the Bathroom Swivel Turnplate onto the required assistive device with the help of your caregiver.



Cleaning & maintenance

- Ensure Bathroom Swivel Turnplate is regularly cleaned with a mild disinfectant
- Bathroom Swivel Turnplate should always be allowed to dry in between use.
- **Never use** if the Bathroom Swivel Turnplate is or appears to be damaged, the product may require replacing.

Product safety

- Your occupational therapist will provide specific recommendations tailored to your circumstances for use of this product. Instructions contained within this document are designed to be used in conjunction with occupational therapist recommendations.
- Always check the Bathroom Swivel Turnplate is stable and secure prior to use.
- **Never use this device without assistance from a caregiver.**
- **Never attempt to move the product while under load.**
- **Never load more than 150kg on the product.**
- **Never use the product if there are any signs of damage.**
- **Never use this product in anyway other way other than the intended use disclosed in this document.**