

Sloped Suction Feet

GUIDE FOR INSTALLATION AND SAFE USE

What are Sloped Suction Feet?

Sloped suction feet are designed to enhance the stability and safety of shower chairs, benches, stools and steps in bathrooms. They are a valuable product for anyone with mobility issues or those requiring extra support, providing peace of mind by increasing a bathroom support device's stability in what can often be a wet and slippery environment.

Sloped Suction Feet are designed for use on shower chairs, benches, stools and steps which will be referred to hereon as the device.

Installation prior to first use

All instructions contained within this document should be used alongside any information provided by your occupational therapist. This product should only ever be used for its intended purpose as described in the document.

- 1. Installation of Sloped Suction Feet:**
 - Remove any previously installed suction feet prior to the installation of your new Q-assist Sloped Suction Feet.
 - Ensure there is enough space around the chair for safe entry and exit.
- 2. Test stability:**
 - Once attached, gently test the stability of your chair by applying slight pressure. It should feel secure and not wobble.
- 3. Safe working load:**
 - Sloped Suction Feet are steel reinforced and have a safe working load of 200kg, never load more than 200kg on the suction feet. Ensure to always check the safe working load of the device the Sloped Suction Feet have been installed on and never exceed the device's maximum safe working load.

Cleaning & maintenance

- Ensure the Sloped Suction Feet are regularly cleaned with a mild disinfectant.
- Sloped Suction Feet should be allowed to dry in between use.
- Fittings should be regularly checked to ensure that they do not become loose after normal operation.
- **Never use** if the Sloped Suction Feet sustains or appears to be damaged, the product may require replacing.



Product safety

- Your occupational therapist will provide specific recommendations tailored to your circumstances for the use of this product. Instructions contained within this document are designed to be used in conjunction with your occupational therapist recommendations.
- Anti slip shower mats are recommended at the entry and exit points.
- Always ensure that the device the suction feet are attached to is stable prior to use.
- Always have everything you need in the shower or bath prior to use.
- **Never attempt to move the product while under load.**
- **Never load more than 200 kg on the product.**
- **Always check the weight rating of the device that the suction feet are installed on and never exceed that device's maximum weight rating.**
- **Never use the product if there are any signs of damage.**
- **Never use this product in any other way other than the intended use as disclosed within this document.**
- **If there is anything that you are unsure about in this document, please consult your occupational therapist.**