

Over Toilet Frame & Commode

SETUP INSTRUCTIONS

Congratulations on the purchase of your Q-assist Over Toilet Frame & Commode.

Type B product suitable for domestic use only.

Included in the box

Please ensure that all parts are present before attempting to setup product.

A	1 x Frame
B	1 x Back Brace
C	1 x Toilet Seat
D	1 x Commode Bucket



Step 1:

Unfold the sides of the toilet frame and stand upright



Step 2:

Attach front cross brace by depressing brass push pins and sliding brace into side frame socket. Repeat on the other side. Be sure brass push pins 'snap' through holes. Attach the back brace by depressing pins and sliding into place.



Step 3:

Lower bucket and toilet seat (or just toilet seat) into the bracket. Attach seat to the back of the frame so that it sits directly on top of the bucket.

