

# Bath Step & Stool

## SETUP INSTRUCTIONS


Congratulations on the purchase of your Q-assist Bath Step & Stool.

Type B product for domestic use only.

### Included in the box


Please ensure that all parts are present before attempting to setup product.

<b>A</b>	1 x Step Base
<b>B</b>	2 x Leg Frames
<b>C</b>	2 x Support Pole
<b>D</b>	2 x Hand Adjustable Screws

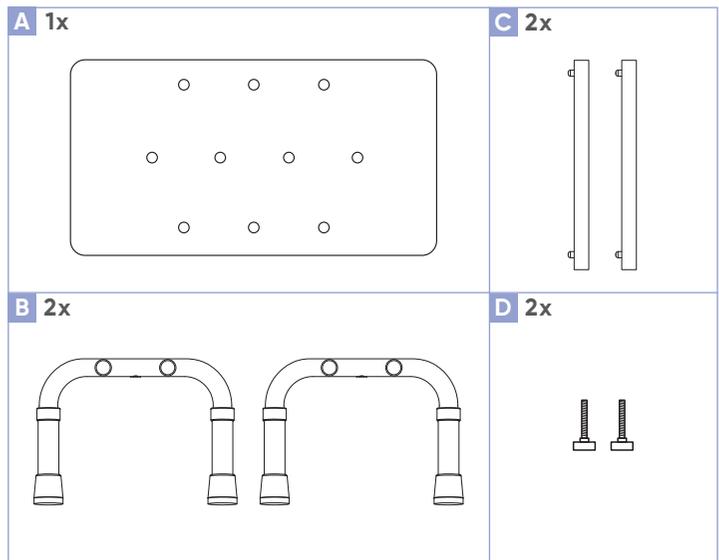


### Step 1:

Install both support poles into the first leg, ensure that the brass button on each of the support poles lock into place.

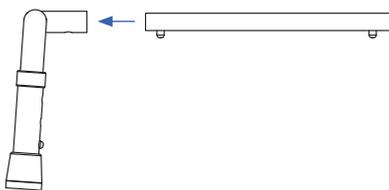

### Step 2:

Repeat step one to install support poles into the second leg.

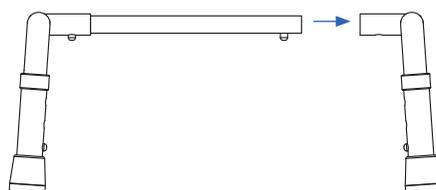

### Step 3:

Fit the frame to the bottom of the base by screwing the hand adjustable screws through the frame into the base on both sides. Do not over tighten, hand tight is sufficient.

