

# Toilet Handrail


## SETUP INSTRUCTIONS

Congratulations on the purchase of your Q-Assist Toilet Handrail.

### Included in the box

Please ensure that all parts are present before attempting to setup product.

<b>A</b>	1 x Fixing Plate
<b>B</b>	1 x Left Hand Arm
<b>C</b>	1 x Right Hand Arm
<b>D</b>	4 x Self-Tapping Screw
<b>E</b>	8 x Screw
<b>F</b>	2 x Washer
<b>G</b>	2 x Integrated Screw Kit

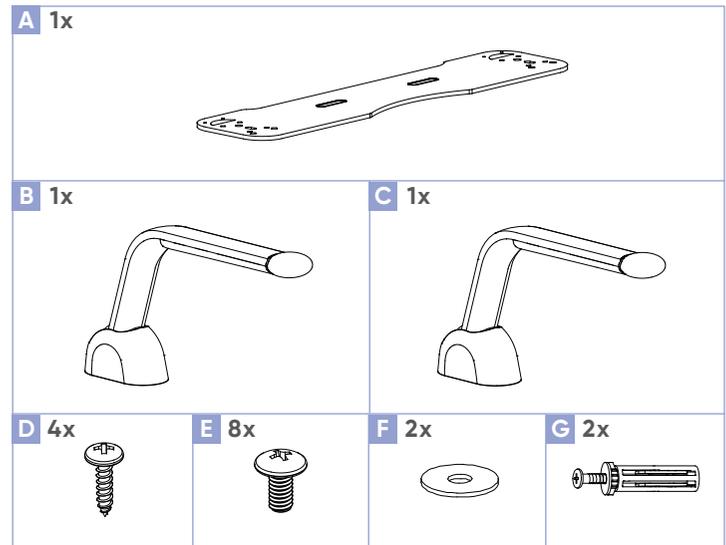

### Please note:

- To prevent interference between the end of the arm rest and the toilet tank, the Fixing Plate must extend more than 60mm beyond the tank of the toilet
- Distance between the toilet installation holes must be between 130mm – 200mm

### INSTALLATION TOOLS REQUIRED




Screwdriver



Soft wet cloth


### Step 1:

To assemble the Toilet Handrail fix the left and right handrail rails to the base fixing plate using the 8 (E) screws.

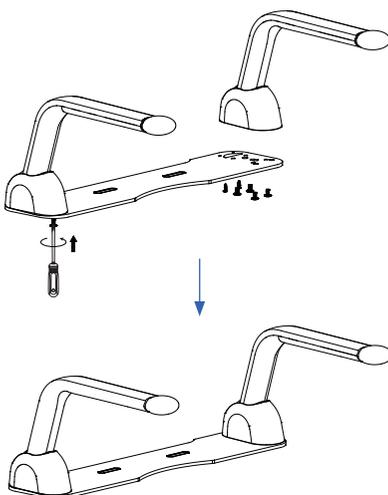


### Step 2:

Remove toilet toilet seat completely including all fixings.

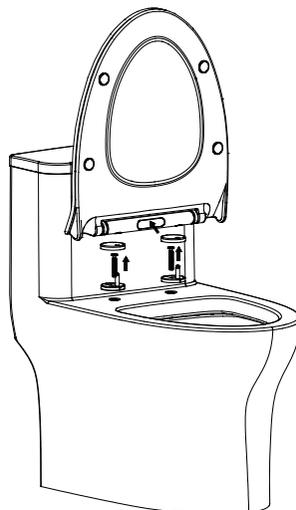


### Step 3:

Install the two integrated screws (G) into the fixing points on your toilet. Never separate the integrated screw.

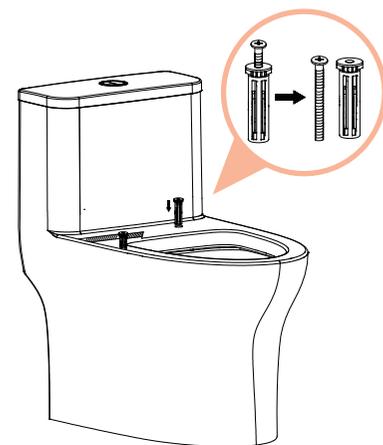


**Step 4:**

Place fixing plate onto the toilet aligning the handrail screw fixings with the holes in the fixing plate.

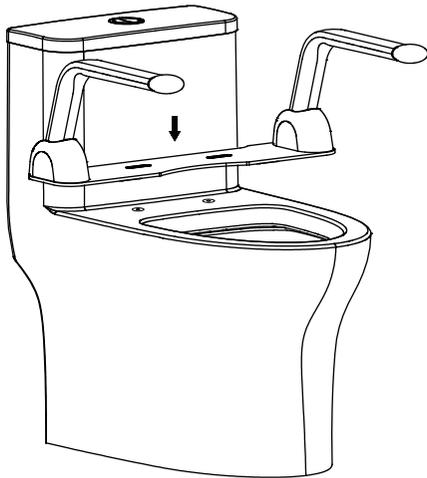


**Step 5:**

Install the fixing plate: install and half-tighten the hinges according to the figure above. (Do not fully tighten the screws)

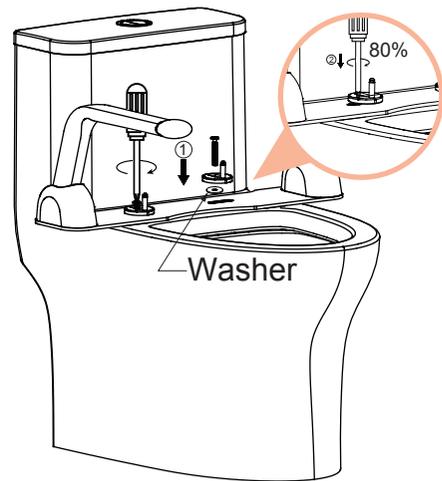

**Step 6:**

Reinstall toilet seat.

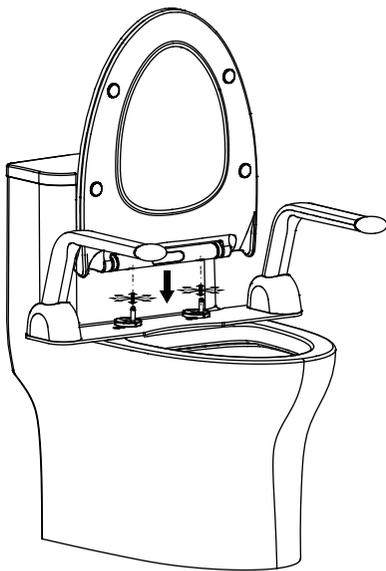

**Step 7:**

Align toilet seat and fully tighten the fixing screws

