

Transfer Chair

SETUP INSTRUCTIONS


Congratulations on the purchase of your Q-assist Transfer Chair.

Type B product for domestic use only.

Included in the box

Please ensure that all parts are present before attempting to setup product.

A	1 x Seat and Frame
B	4 x Chair Legs
C	1 x Backrest
D	2 x Backrest Tubes
E	1 x Arm Rest

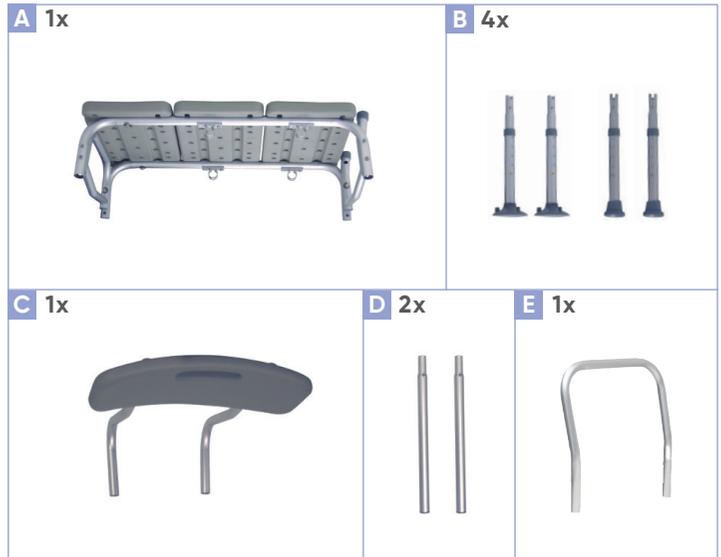

Step 1:

Lay the seat and frame upside down, on a flat surface. Attach one of the legs with the large suction cup by depressing one of the legs with the large suction cup by depressing the push buttons and inserting them into the frame. Repeat for the second leg with the large suction cup. Ensure button locks into place.



Step 2:

Attach the first leg with rubber base by depressing the push buttons and inserting into the frame. Repeat for the second leg with rubber base. Ensure button locks into place.




Step 3:

Turn the Transfer Chair the right way up. Attach the armrest by depressing the push buttons and sliding it into the arm receptacle, on the end of the seat and frame. Ensure the armrest arcs outwards, not inwards.



Step 4:

To assemble the backrest, insert the pole into the backrest support by depressing the push buttons and sliding it into the receptacle. Repeat for the second pole.

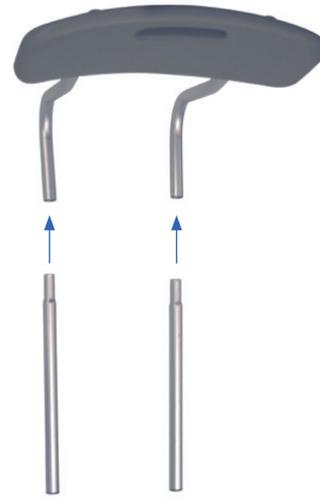

Step 5:

Insert the backrest into the base of the Transfer Chair by depressing the push buttons and sliding the supports through both holes in the bottom of the frame.



Note: Backrest must go through both sets of receptacles to become secure. The backrest is reversible to accommodate left or right tub entry.

